

HICIDE CONTRA

Fast-acting disinfectant for manual processing of handpieces, contra angles & turbines

HICIDE CONTRA is a ready-to-use disinfectant for manual processing of handpieces, contra angles and turbines. Ultra-fast-acting formulation besides broad-spectrum antimicrobial activity, combined with its excellent stability and environmentally friendly characteristics, have made it the best choice for disinfection of alcohol-resistant instruments

Characteristics

- Ready-to-use disinfectant
- Broad-Spectrum antimicrobial activity
- Contains no dangerous phenols, aldehydes or chlorine

Product Specification

Color/Form	Colorless to pale yellow clear solution
pH	2.5-5.5
Scent	Alcohol-like odor
Shelf Life	24 months



Special for
Infection Control
in Dental Settings

Ingredients

100 g HICIDE CONTRA contains:

- 53.7 g Ethanol
- 19 g n-Propanol

Auxiliary ingredients:

- Surfactant
- Corrosion inhibitors
- Solubilizer
- Essence
- Deionized water

Microbiological Effectiveness

Meets European Standards

Bactericidal Activity

According to EN 13727 (contact time: 1 minute) *Staphylococcus. aureus*/ ATCC 6538
Escherichia. coli/ ATCC 10536
Pseudomonas. aeruginosa / ATCC 15442
Enterococcus. hirae/ ATCC 10541

According to EN 14561 (contact time: 5 minutes) *Staphylococcus. aureus*/ ATCC 6538
Pseudomonas. aeruginosa / ATCC 15442
Enterococcus. hirae/ ATCC 10541

Yeasticidal Activity

According to EN 13624 (contact time: 1 minute) *Candida. albicans*/ ATCC 10231

According to EN 14562 (contact time: 5 minutes) *Candida. albicans*/ ATCC 10231

Fungicidal Activity

According to EN 13624 (contact time: 5 minutes) *Aspergillus. niger*/ ATCC 16404

According to EN 14652 (contact time: 5 minutes) *Aspergillus. niger*/ ATCC 16404

Virucidal Activity

According to EN 14476 (contact time: 3 minutes) *Herpes simplex virus (HSV-1)*
(contact time: 5 minutes) *Adenovirus type 5*



Directions for use

Connect adapter to the product bottle. Fully insert adapter into the connection end of handpiece or turbine. Hold product bottle upside down. Keep squeezing the trigger until the liquid is coming out from the instrument head. Wait for 5 minutes. Then, hold product bottle upright and cover the instrument with a single use cloth. Keep squeezing the trigger several times to flush out the solution and to blow-dry the instrument. Use single-use cloth for the outer cleaning of handpiece or turbine.

Packagings

3 canisters per carton,

5 L each

20 bottles per carton,

500 mL each

Hazard Rating

Ready-to-use

NFPA

Health	1
Flammability	3
Reactivity	1



Warning

- Keep out of reach of children and unaware persons.
- Highly flammable. Keep away from heat, spark and high-voltage electric current.
- Store between + 5 °C and + 35 °C and away from direct sunlight.



Certifications

- >> ISO 9001: 2015
- >> ISO 13485: 2016
- >> GMP
- >> FSC



HISEPT[®]
HIGH-LEVEL SAFETY & PROTECTION

Middle East Office
Behban Pharmed Lotus Co.
Address : Unit 113, 12 Floor, Bldg. B, Kaveh
Complex, Azadi Ave., Tehran, Iran
Tel: (+9821) 66581294-5/66581269

info@hisept.com

World Locations: Düsseldorf, Kuala Lumpur, Istanbul, Tehran, Baghdad