

NEO

Ready-to-use disinfectant for medical equipment and surfaces

HISEPT NEO is a ready-to-use disinfectant for medical equipment and surfaces. Broad-spectrum Virucidal, bactericidal and yeasticidal properties, combined with its excellent stability and eco-friendly features, have made this product the best choice for disinfection of all alcohol-sensitive medical equipment while being an ideal surface disinfectant.

Characteristics

- Ready-to-use disinfectant, ideal for alcohol-sensitive equipment
- Contains no dangerous phenols, aldehydes or chlorine
- User-friendly design of sprayer head
- Excellent cleaning efficiency

Product Specification

Color/Form	Colorless to yellow clear solution
pH	7-10
Scent	Pleasant odor
Shelf Life	24 months



THIS IS A HISEPT PRODUCT, WITH
**HIGH-LEVEL
SAFETY &
PROTECTION**

Ingredients

100 g HISEPT NEO contains:

- 0.37 g N-(3-aminopropyl)-N-dodecylpropane-1, 3-diamine

Auxiliary ingredients:

- Ethoxylated alcohol
- pH-Adjusting agent
- Essence
- Deionized water

Microbiological Effectiveness

Meets European Standards

Bactericidal Activity

According to EN 13727 (contact time: 5 minutes) Staphylococcus. aureus/ ATCC 6538
Enterococcus. hirae/ ATCC 10541
Pseudomonas. aeruginosa/ ATCC 15442

According to EN 13697 (contact time: 5 minutes) Staphylococcus. aureus/ ATCC 6538
Enterococcus. hirae/ ATCC 10541
Pseudomonas. aeruginosa/ ATCC 15442
Escherichia. coli/ ATCC 10536

Yeasticidal Activity

According to EN 13624 (contact time: 5 minutes) Candida. albicans/ATCC 10231

According to EN 13697 (contact time: 5 minutes) Candida. albicans/ATCC 10231

Virucidal Activity

According to EN 14476 (contact time: 3 minutes) Adenovirus type 5
Herpes simplex virus (HSV-1)



Directions for use

Spray directly to the surface or device or spray to a clean linen cloth. Wipe and wait for at least 3 minutes.

Compatibility

HISEPT NEO is compatible with most materials such as stainless steel, copper, plastic, rubber and silicon.


2/2 Latest revision: Dec 1, 2023

Packagings

6 bottles per carton,	1 L each
20 bottles per carton,	500 mL each
30 bottles per carton,	250 mL each
48 bottles per carton,	125 mL each

Hazard Rating

Ready-to-use	NFPA
Health	1
Flammability	0
Reactivity	0

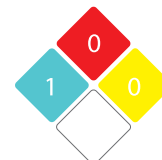

Warning

- Keep out of reach of children and unaware persons.
- Store between + 5 °C and + 35 °C and away from direct sunlight.



Certifications

- >> ISO 9001: 2015
- >> ISO 13485: 2016
- >> GMP
- >> FSC



Middle East Office
Behban Pharmed Lotus Co.
Address : Unit 113, 12 Floor, Bldg. B, Kaveh
Complex, Azadi Ave., Tehran, Iran
Tel: (+9821) 66581294-5/66581269

info@hisept.com

World Locations: Düsseldorf, Kuala Lumpur, Istanbul, Tehran, Baghdad