

# HICIDE S-125

## Ready-to-use, alcohol-free disinfectant for medical equipment and surfaces

HICIDE S-125 is a ready-to-use and rapid disinfectant for medical equipment and surfaces. Broad-spectrum bactericidal, fungicidal, yeasticidal and virucidal properties, combined with its excellent stability and eco-friendly features, have made this product the best choice for disinfection of hard and porous surfaces.

## Characteristics

- Compatible with most hard and porous surfaces
- Broad-spectrum antimicrobial effect
- Environmentally friendly
- Non-rinsing formula
- One-step cleanser/disinfectant

## Product Specification

Color/Form	Clear & colorless solution
pH	2-4
Scent	Characteristic odor
Shelf Life	18 months



THIS IS A HISEPT PRODUCT, WITH  
**HIGH-LEVEL  
SAFETY &  
PROTECTION**

## Ingredients

100 g HICIDE S-125 contains:

- 1.5 g Hydrogen peroxide
- 0.25 g Didecyltrimethylammonium chloride

Auxiliary ingredients:

- Chelating agent
- Stabilizer
- Corrosion inhibitors
- Deionized water

## Microbiological Effectiveness

### Meets European Standards

#### Bactericidal Activity

According to EN 13727 Staphylococcus aureus/ ATCC 6538  
(contact time: 5 minutes) Enterococcus hirae/ ATCC 10541  
Pseudomonas aeruginosa/ ATCC 15442

#### Yeasticidal Activity

According to EN 13624 Candida albicans/ATCC 10231  
(contact time: 5 minutes)

#### Fungicidal Activity

According to EN 13624 Aspergillus niger/ATCC 16404  
(contact time: 5 minutes)

#### Virucidal Activity

According to EN 14476 Adenovirus  
(contact time: 1 minute) Herpes simplex virus (HSV-I)



## Directions for use

Spray directly to the surface or spray to a clean linen cloth. Wipe and wait for at least 5 minutes.

## Compatibility

HICIDE S-125 is compatible with most materials such as stainless steel, plastic, poly propylene, copper, PVC and silicon.

2/2 Latest revision: Dec 1, 2023


## Packagings

3 canisters per carton,  
20 bottles per carton,

5 L each  
500 mL each

## Hazard Rating

Ready-to-use	NFPA
Health	1
Flammability	0
Reactivity	0

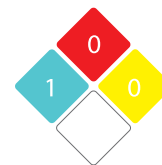

## Warning

- Keep out of reach of children and unaware persons.
- Store between + 5 °C and + 35 °C and away from direct sunlight.



## Certifications

- >> ISO 9001: 2015
- >> ISO 13485: 2016
- >> GMP
- >> FSC



**HISEPT**<sup>®</sup>  
HIGH-LEVEL SAFETY & PROTECTION

Middle East Office  
**Behban Pharmed Lotus Co.**  
Address : Unit 113, 12 Floor, Bldg. B, Kaveh  
Complex, Azadi Ave., Tehran, Iran  
Tel: (+9821) 66581294-5/66581269

info@hisept.com

**World Locations:** Düsseldorf, Kuala Lumpur, Istanbul, Tehran, Baghdad