

SEPTONEAT

Concentrated broad-spectrum disinfectant for medical surfaces


HISEPT SEPTONEAT is a concentrated cleaner/disinfectant based on alkylamine combined with powerful surfactants, creating good cleaning efficiency. Besides, HISEPT SEPTONEAT has excellent compatibility with most materials used in healthcare centers. It is recommended for routine cleaning and disinfection of hard surfaces.

Characteristics

- Broad-spectrum antimicrobial activity
- Excellent cleaning performance
- Without sticky residues
- Contains no dangerous phenols, aldehydes or chlorine

Product Specification

Color/Form	Colorless to pale yellow clear solution
pH	7-10
Scent	Characteristic odor
Shelf Life	24 months
Solubility (in water)	Completely soluble

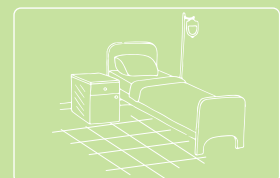

THIS IS A HISEPT PRODUCT, WITH
**HIGH-LEVEL
SAFETY &
PROTECTION**

Ingredients

100 g HISEPT SEPTONEAT contains:

- 7.5 g N-(3-aminopropyl)-N-dodecylpropane-1, 3-diamine

Auxiliary ingredients:

- Ethoxylated alcohol
- pH-Adjusting agent
- Deionized water

Microbiological Effectiveness

Meets European Standards

Bactericidal Activity

According to EN 1276 *Pseudomonas. aeruginosa*/ ATCC 15442
(contact time: 5 minutes
use dilution: 2%)
Escherichia. coli/ ATCC 10536

According to EN 13727 *Staphylococcus. aureus*/ ATCC 6538
(contact time: 5 minutes
use dilution: 2%)
Enterococcus. hirae/ ATCC 10541
Pseudomonas. aeruginosa/ ATCC 15442

According to EN 13697 *Staphylococcus. aureus*/ ATCC 6538
(contact time: 5 minutes
use dilution: 2%)
Enterococcus. hirae/ ATCC 10541
Pseudomonas. aeruginosa/ ATCC 15442
Escherichia. coli/ ATCC 10536

Yeasticidal Activity

According to EN 13624 *Candida. albicans*/ATCC 10231
(contact time: 5 minutes
use dilution: 2%)

According to EN 13697 *Candida. albicans*/ATCC 10231
(contact time: 5 minutes
use dilution: 2%)

Virucidal Activity

According to EN 14476 *Herpes simplex virus (HSV-1)*
(contact time: 5 minutes
use dilution: 2%)

Packagings

3 canisters per carton,


5 L each

6 bottles per carton,

1 L each

Hazard Rating

Concentrate	NFPA
Health	2
Flammability	0
Reactivity	0

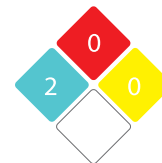

Warning

- Keep out of reach of children and unaware persons.
- Store between + 5 °C and + 35 °C and away from direct sunlight.



Certifications

- >> ISO 9001: 2015
- >> ISO 13485: 2016
- >> GMP
- >> FSC



Middle East Office
Behban Pharmed Lotus Co.
Address : Unit 113, 12 Floor, Bldg. B, Kaveh
Complex, Azadi Ave., Tehran, Iran
Tel: (+9821) 66581294-5/66581269

info@hisept.com

World Locations: Düsseldorf, Kuala Lumpur, Istanbul, Tehran, Baghdad

HISEPT[®]
HIGH-LEVEL SAFETY & PROTECTION

Directions for use

The concentrate shall be diluted before use with water to 2 %. Apply the solution onto the surfaces, wait solution remain on surface for at least 5 minutes. Then mop to clean and disinfect surfaces.

Compatibility

HISEPT SEPTONEAT is compatible with most materials such as stainless steel, copper, plastic, rubber and silicon.